PRODUCTINFORMATIE ABDONIMAL CORSET

Dutch

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PRODUCT INFORMATION ABDOMINAL CORSET


English

Description:

The Abdominal Band is a 26 centimeter wide, elastic support belt for compression and support of the abdominal area (stomach). The Velcro fastener makes the brace easy to use.

Sizes:

SIZE	BELLY SIZE	ART.NR.
S	75 - 85 cm	PO 31420 S
M	85 - 95 cm	PO 31420 M
L	95 - 105 cm	PO 31420 L
XL	105-115 cm	PO 31420 XL
XXL	115 - 125 cm	PO 31420 XXL

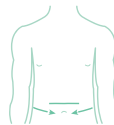Indications: Postoperative (abdominal region), peritoneal injury, postnatal interventions, support of the weakened abdominal muscles

Manual:

1. Take both ends and fasten the bandage from the back towards the stomach with the Velcro fastener.
2. Make sure that the bandage is tight enough that it provides support and at the same time does not hinder breathing.

Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with your doctor's advice and instructions. The product should be installed by a trained professional the first time.

In cases where the product is recommended by the

doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided.

This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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