## PRODUCTINFORMATIE BUDDY SPLINT

Dutch

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**Indicaties:** Reuma, vingerfracturen  
Kneuzingen, triggervingers

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## PRODUCT INFORMATION BUDDY SPLINT

English

**Description:** The Buddy Splint is a comfortable finger splint for quick, comfortable and effective protection of the finger. The splint is easy to put on and the straps are made of soft foam. The product is latex-free and has an anti-slip effect for ideal position retention. It can be used not only on the fingers but also on the toes.

### Sizes:

SIZES	FINGER SIZE	ART.NR.
ONZE SIZE		SP EZB

**Indications:** Rheumatism, finger fractures  
Bruises, trigger fingers

### Manual:

- Slide the cuff over the affected finger
- Skip the long section above or below the adjacent NOT affected finger.
- Then fold the long part over the affected finger and stick the Velcro fastener on the cuff

### Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with the advice and instructions of your physician. The product should be installed by a trained professional the first time.

In cases where the product is recommended by the doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided.

This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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