PRODUCTINFORMATIE EPI COMFORT PLUS

Dutch

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PRODUCT INFORMATION EPI COMFORT PLUS

English

Description: Pro Orthic Epi Comfort Plus is an anatomically woven elbow bandage with medial and lateral silicone pads and specially knitted areas in the elbow bend. Provides a proprioceptive effect and allows a faster healing process. The Velcro closure provides extra support and compression.

Sizing:

MAAT	ELBOW SIZE	ART.NR.
S	22 - 24 cm	PO 31307 S
M	24 - 26 cm	PO 31307 M
L	26 - 28 cm	PO 31307 L
XL	28 - 30 cm	PO 31307 XL
XXL	30 - 32 cm	PO 31307 XXL

Indications: Epicondylalgia lateralis (formerly epicondylitis lateralis, also called tennis elbow), Epicondylalgia medialis (golfer's elbow, Overuse injuries, Osteoarthritis)

Manual:

- Slide the brace over the elbow.
- Place the pad on the part of the muscle belly closest to the elbow, making sure the tendon remains free. (Usually the pad is placed about 5 centimeters below the hinge point of the elbow)
- Close the Velcro fasteners to achieve the desired compression. Make sure that the brace is not too tight. Too much compression can cause reduced blood flow.

Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with the advice and instructions of your physician. The product should be installed by a

trained professional the first time.

In cases where the product is recommended by the doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided.

This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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