PRODUCTINFORMATIE GENU AIRTEX

Dutch

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English

Description:

The Pro Orthic Genu Airtex is a knee brace for light knee support. Helps relieve pain and stabilize the patella.

Sizing:

SIZE	ART.NR.
ONE SIZE	PO 31603-ES

Indications: Mild meniscus or cruciate ligament sprains, weak or overstretched knee, osteoarthritis, arthritis

Manual:

1. Extend the knee and place the silicone ring around the kneecap.
2. Attach the middle strap first. Provide adequate compression, but avoid getting too tight around the back of the knee.
3. Then fasten the top strap.
4. Finally the lower strap is attached. With these two straps, make sure that the straps are not too tight around the leg.

Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with the advice and instructions of your physician. The product should be installed by a

trained professional the first time.

In cases where the product is recommended by the doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided.

This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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