PRODUCTINFORMATIE HOT & COLD BANDAGE KNEE

Dutch

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PRODUCT INFORMATION HOT & COLD KNEEBANDAGE

English

Description: Hot & Cold bandage for cooling (acute) knee complaints or warming the knee in case of stiffness and joint pain. The bandage is easy to use and has optimal wearing comfort because of the material and design.

Instruction for refrigeration:

1. Remove the gel pack from the bandage and place it in the freezer for at least two hours; or until the desired temperature is reached.
2. Check that the temperature of the gel pack is comfortable before applying the bandage.

Warm up instruction:

1. The gel pack should be at room temperature before being heated.
2. Remove the gel pack from the bandage and place it in a damp cloth or paper towel, then place it in the center of a microwave. Make sure that all additional functions of the microwave are not switched on. (such as grill function)
3. Use the table below to determine the warm-up time. Check that the temperature of the gel pack is comfortable before applying the bandage.
4. If the correct temperature has not yet been reached: Check beforehand whether the heat is equal over the entire gel packing and distribute the heat if necessary. Heat in steps (see table) until the desired temperature is reached.

Indications: Cooling: Knee pain, sports injuries, swelling, bruising and muscle aches.

Heat: stiffness, muscle pain, sports injuries and swelling.

Installation instruction:

1. Place the round opening on the kneecap. Make sure the Pro Orthic logo is on the top of the knee.
2. Fasten the straps above and below the kneecap with the Velcro fastener and ensure sufficient tension so that the brace stays in place.

ABILITY	WARM UP TIME	ADDITIONAL WARM UP TIME
800 WATT	15 SEC.	10 SEC.
1000 WATT	10 SEC.	5 SEC.

Important information and warnings:

The product should be used in accordance with the advice and instructions of your physician. The product should be installed by a trained professional the first time.

The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule

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