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PRODUCT INFORMATION HOT & COLD WRIST BAND

English

Description: The Pro Orthic Hot & Cold wrist bandage is a bandage for cooling in case of (acute) wrist complaints or heating the wrist joint in case of stiffness and joint pain. The bandage is easy to use and has optimal wearing comfort because of the material and design.

Instruction for refrigeration:

1. Remove the gel pack from the bandage and place it in the freezer for at least two hours; or until the desired temperature is reached.

2. Check that the temperature of the gel pack is comfortable before applying the bandage.

Warm up instruction:

1. The gel pack should be at room temperature before being heated.

2. Remove the gel pack from the bandage and place it in a damp cloth or paper towel, then place it in the center of a microwave. Make sure that all additional functions of the microwave are not switched on. (such as grill function)

3. Use the table below to determine the warm-up time. Check that the temperature of the gel pack is comfortable before applying the bandage.

4. If the correct temperature has not yet been reached: Check beforehand whether the heat is equal over the

Indications: *Cold compress* - rheumatism, swelling, bruise/sprain, stiffness, reduction of inflammation and pain. *Warm compress* - stiffness, reduction of pain, muscle spasms.

Installation instruction:

1. Place the opening on the thumb side and insert the thumb through the opening.

2. Wrap the bandage around the wrist and close the Velcro

ABILITY	WARM UP TIME	ADDITIONAL WARM UP TIME
800 WATT	15 SEC.	5 SEC.
1000 WATT	10 SEC.	5 SEC.

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