PRODUCTINFORMATIE JOINT 8

Dutch

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PRODUCT INFORMATION JOINT 8

English

Description: Joint 8 Finger Splint is the ideal tool to stabilize or immobilize the small joints of the fingers. The Joint 8 Finger Splint is designed to stabilize or immobilize the PIP joint or DIP joint. The splint is also made with 1.6mm translucent plastic without seams. The edges are rounded to provide more comfort.

Sizes:

SIZES	FINGER SIZE	ART.NR.
2 - 15	4.3 CM - 8.2 CM	SP W-51008

Indications: Deformities of the Finger, Trigger Finger and Trigger Thumb, Lateral Deviation, Arthritis, Finger Fractures or Hypermobility (Ehlers-Danlos Syndrome)

Manual:

1. With a simple turn of the splint, any Joint-8 Finger Splint can be used to treat the above injuries.

Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with the advice and instructions of your physician. The product should be installed by a trained professional the first time.

In cases where the product is recommended by the doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided.

This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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