## PRODUCTINFORMATIE LUMBO BASIC

Dutch

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## PRODUCT INFORMATION LUMBO BASIC

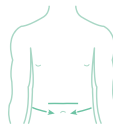
English

**Disclaimer:** Pro Orthic Lumbo Basic is a corset with support bands made of elastic compression fabric, closed at the front with Velcro. The corset provides compression and support and stabilizes the spine. Consists of two flexible steel beams, four plastic ribs at the front and rear and a belt system.

### Sizing:

SIZE	BELLY SIZE	ART.NR.
S	75 - 85 cm	PO 31260-K S
M	85 - 95 cm	PO 31260-K M
L	95 - 105 cm	PO 31260-K L
XL	105-115 cm	PO 31260-K XL
XXL	115 - 125 cm	PO 31260-K XXL

**Indications:** Spondylolisthesis, Inflammation of the spinal joints, Degenerative changes

### Manual:

- Make sure the steel ribs are placed in the center of the lower back. The label should be on top.
- Close the lower, larger band with the Velcro on the stomach.
- Then take the narrower band and place it on the stomach and hold it until the narrower band can also be attached to it from the other side with the Velcro fastener.

### Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with the advice and instructions of your physician. The product should be installed by a trained professional the first time.

In cases where the product is recommended by the doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided. This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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