PRODUCTINFORMATIE MALLEO COMFORT

Dutch

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PRODUCT INFORMATION MALLEO COMFORT

English

Discription: The Malleo Comfort is a supportive and soothing support for sore ankles made of breathable, climate-friendly knitted material. Anatomically shaped with specially knitted parts on the flexion zone of the ankle. Provides compression, improves circulation, relieves pain and prevents stiffness.

Sizing:

SIZE	SIZE ANKLE	ART.NR.
S	19 - 21 cm	PO 31704 S
M	21 - 23 cm	PO 31704 M
L	23 - 25 cm	PO 31704 L
XL	25-27 cm	PO 31704 XL
XXL	27 - 29 cm	PO 31704 XXL

Indications: Ankle sprain, Bruised ankle. Slight instability, Return to sport after ankle injuries

Manual:

- Pull the bandage around your foot like a sock, with the text legible from the front.
- Then pull the sock into a position where it can be worn comfortably.

Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with the advice and instructions of your physician. The product should be installed by a trained professional the first time.

In cases where the product is recommended by the doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided. This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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