PRODUCTINFORMATIE MANU STABLE

Dutch

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PRODUCT INFORMATION MANU STABLE

English

Description: The Manu Stable is a contoured, adjustable aluminum wrist rest with plastic ribs on both sides for a better grip. Made of breathable fabric that provides better ventilation of the skin and high patient comfort.

Sizing:

SIZE	WRIST SIZE	ART.NR.
S	14 - 19 cm	PO 3119-F S
M	19 - 25 cm	PO 3119-F M

Indications: CTS (carpal tunnel syndrome), Recovery after fracture, Immobilization wrist, Bruise/Sprain

Manual:

- Open all Velcro fasteners and place the orthosis around the wrist.
- Make sure that the anatomically shaped splint lies comfortably in the palm of your hand.
- Then close the Velcro straps and ensure sufficient compression. Make sure the clamp is not too tight. Excessive compression can lead to decreased blood flow.

Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with the advice and instructions of your doctor. The product should be installed by a trained professional the first time.

In cases where the product is recommended by the doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided.

This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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