PRODUCTINFORMATIE NIGHT SOCK

Dutch

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PRODUCT INFORMATION NIGHT SOCK

English

Description: The Pro Orthic Night Sock is a night splint to promote the recovery of heel spurs, tendonitis (plantar fasciitis) and/or Achilles tendon complaints. The new design provides improved and optimal wearing comfort, while the toes retain freedom of movement. With an adjustable strap, an optimal stretch is obtained.

Sizing:

SIZE	HEEL SIZE	ART.NR.
S - W	< 40 cm	PO NS 100W S
L - W	> 40 cm	PO NS 100W L
S - B	< 40 cm	PO NS 100B S
L - B	> 40 cm	PO NS 100B L

Indications: (Acute) Back pain, Muscle pain, Spasm, Joint pain

Manual:

- Put on the sock with the buckles on the sides of the shin, just below the kneecap. The lower strap is placed on the ball of the foot, just below the toes.
- Close the top strap. Make sure the band is tight so that the sock does not fall down during the night while sleeping.
- Pass the Velcro straps on the side through the buckles and lift the toes towards the chest with the knee straight, so that the foot is at 90 degrees. Hold this position and at the same time tighten the Velcro straps on the side.
- When a slight stretch is felt on the calf and/or under the sole of the foot, the sock has been fitted correctly. "We prefer to wear the sock every night, this is the best way to promote recovery."

Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with your doctor's advice and instructions. The product should be installed by a trained professional the first time.

In cases where the product is recommended by the doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided.

This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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