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PRODUCT INFORMATION RHEUMATISM GLOVES

English

Description: Rheumatism gloves provide mild compression and the warmth of this arthritis glove improves blood circulation. In addition, the glove can be used to reduce swelling and promote the possible healing process. These arthritis gloves are made of soft breathable cotton Lycra® which allows the skin to breathe and the hands stay dry. The open fingertips give complete freedom of feel, touch and grip.

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Indications: Rheumatism, Arthritis and Tendinitis, Carpal Tunnel Syndrome, Osteoarthritis

Manual:

1. To determine the correct size, measure the width of the hand at the knuckles.

Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with the advice and instructions of your physician. The product should be installed by a trained professional the first time.

In cases where the product is recommended by the doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided.

This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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PRODUKTINFORMATION RHEUMAHANDSCHUHE

Deutsch

die Beschreibung: Rheumatism gloves provide mild compression and the warmth of this arthritis glove improves blood circulation. In addition, the glove can be used to reduce swelling and promote the possible healing process. These arthritis gloves are made of soft breathable cotton Lycra® which allows the skin to breathe and the hands stay dry. The open fingertips give complete freedom of feel, touch and grip.

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