PRODUCTINFORMATIE TFCC BRACE

Dutch

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PRODUCT INFORMATION TFCC BRACE

English

Description: The Pro Orthic TFCC brace is a wrist bandage for reducing Triangular FibroCartilage Complex (TFCC) related complaints. Due to the design of the brace, it is ensured that there is no pressure on the ulna head and optimal wearing comfort is created.

Sizing:

SIZE	SIDE	ART.NR.
ONE SIZE	LEFT	PO 319001
	RIGHT	

Indications: Triangular FibroCartilage Complex (TFCC)

Manual:

1. Open the Velcro fasteners
2. The open side of the brace is placed on the back of the hand.
3. Provide adequate compression by closing the Velcro fastener. Make sure that the brace is not too tight. Too much compression can cause reduced blood flow.

Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with your doctor's advice and instructions. The product should be installed by a trained professional the first time.

In cases where the product is recommended by the doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided.

This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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